



Getting Started With SonoSim

For Learners

The SonoSim Ultrasound Training Solution is a proven-effective ultrasound training platform that delivers:



24/7 access to online courses by leading ultrasound educators



On-demand scanning of **real-patient pathology**



A **built-in tutor** to assist with **image acquisition & interpretation**



A **portfolio of your activity** to demonstrate ultrasound knowledge & proficiency

Next Steps to Start Using SonoSim!

Take a SonoSim Course to Build a Strong Didactic Foundation

Two Ways to Access the SonoSim Course Library *(Note: Must have internet connection)*

From Your Internet Browser

1. Go to sonosim.com/login.
2. **Login into the SonoSim Dashboard** with your user credentials provided in your welcome email.
3. **Select "SonoSim Course Library"** from the Dashboard.

Through the Software

1. **Double-click the SonoSim icon** on your computer with the Probe plugged in.
2. **Log in** with your user credentials provided in your welcome email.
3. **Select "SonoSim Courses"** from the Main Menu.



Not sure where to start? Try the Fundamentals of Ultrasound Course to get acquainted with SonoSim Courses.

Scan Cases in the SonoSimulator® to Develop Image Acquisition & Interpretation Skills

1. **Connect to the internet** and **plug in your SonoSim Probe** to the computer's USB port
2. **Double-click the SonoSim icon** on the computer to launch the SonoSim Ultrasound Training Solution
3. **Log in with your user credentials** from your welcome email and select **"SonoSimulator"**
4. For the Probe to scan optimally, please follow the directions to **calibrate**
5. **Use the Case List** in the bottom left to navigate to different patient cases

Not sure where to start? Try the SonoSim SkillBox to build requisite psychomotor skills.

Case List - Select the case you want to scan

Layers - Remove body layers to view how the ultrasound beam transects underlying anatomy

Probe Guide - Follow a virtual Probe to obtain ultrasound image acquisition guidance


Findings - Get ultrasound image interpretation assistance by an expert instructor

Freeze, Save, Annotate - Use these steps to save images & data to Performance Tracker

User Guide

For Groups: Use Performance Tracker to Build a Portfolio of Your Ultrasound Proficiency

1. Go to sonosim.com/login
2. Log into the SonoSim Dashboard with your user credentials provided in your welcome email
3. Select "SonoSim Performance Tracker"

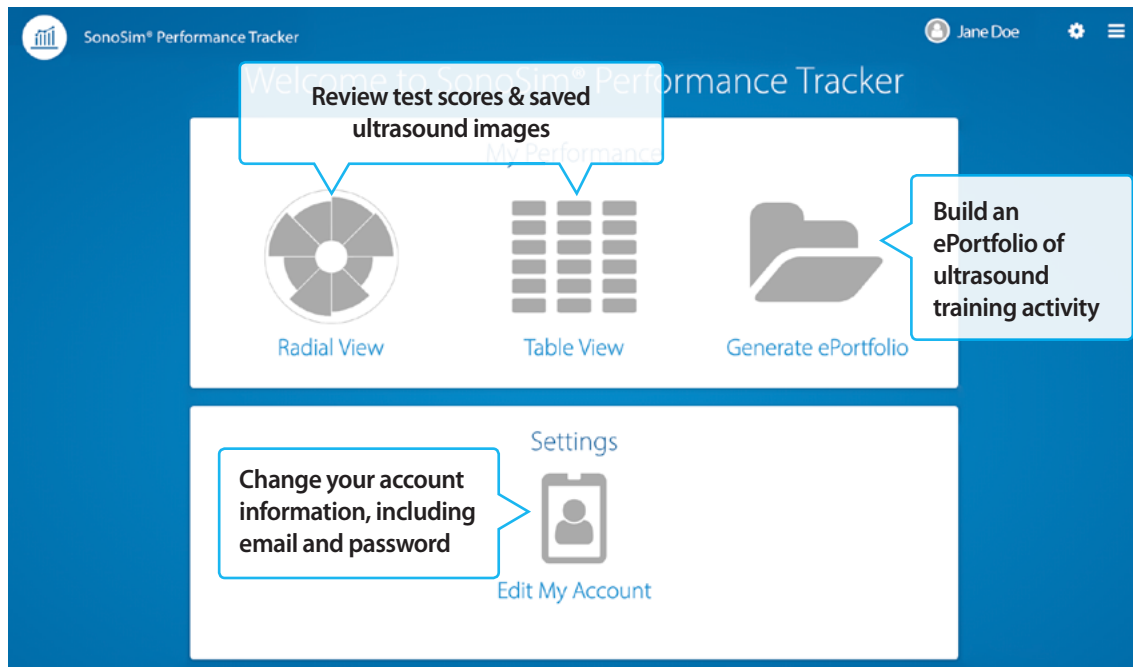

Not sure where to start? Explore "Generate ePortfolio" to build a PDF of your ultrasound images and Module progress.

▶ For a video tour of product features, please visit sonosim.com/videos.


Questions? Need a personal demo?

Our Member Support Team is ready to assist!

For a personal demo or help using the features, please contact support@sonosim.com with the subject line "Product Overview" or call 855.873.7666 (M-F, 6am-6pm PT)

The SonoSim App

Exclusive for Members

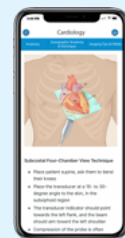

The Easiest Way to Learn Ultrasonography®

