



# Getting Started With SonoSim

## For Learners

The SonoSim Ultrasound Training Solution is a proven-effective ultrasound training platform that delivers:



**24/7 access to online courses** by leading ultrasound educators



On-demand scanning of **real-patient pathology**



A **built-in tutor** to assist with **image acquisition & interpretation**



A **portfolio of your activity** to demonstrate ultrasound knowledge & proficiency

## Next Steps to Start Using SonoSim!

### Take a SonoSim Course to Build a Strong Didactic Foundation

With your login credentials from your welcome email, access SonoSim Courses in three ways (*Note: You must be connected to the internet*):

1. Log in to [sonosim.com/login](https://sonosim.com/login) or
2. **Plug in your SonoSim Probe** & click the SonoSim icon on your computer or
3. **Download the SonoSim App** on your phone or tablet



**Not sure where to start?** Try the *Fundamentals of Ultrasound Course* to get acquainted with SonoSim Courses.

### Scan Cases in the SonoSimulator® to Develop Image Acquisition & Interpretation Skills

1. **Connect to the internet** and **plug in your SonoSim Probe** to the computer's USB port
2. **Double-click the SonoSim icon** on the computer to launch the SonoSim Ultrasound Training Solution
3. **Log in with your user credentials** from your welcome email and select **"SonoSimulator"**
4. For the Probe to scan optimally, please follow the directions to **calibrate**
5. **Use the Case List** in the bottom left to download and navigate to real-patient cases

**Not sure where to start?** Try the *SonoSim SkillBox* to build requisite psychomotor skills.

Adv. FoCUS: Part I  
Case 2  
Parasternal

**Case List** - Select cases to download and scan

**Layers** - Remove body layers to view how the ultrasound beam transects underlying anatomy

**Probe Guide** - Follow a virtual Probe to obtain ultrasound image acquisition guidance


**Findings** - Get ultrasound image interpretation assistance by an expert instructor

**Freeze, Save, Annotate** - Use these steps to save images & data to Performance Tracker

**User Guide**

## For Groups: Use Performance Tracker to Build a Portfolio of Your Ultrasound Proficiency

1. Go to [sonosim.com/login](https://sonosim.com/login)
2. Log into the SonoSim Dashboard with your user credentials provided in your welcome email
3. Select “SonoSim Performance Tracker”

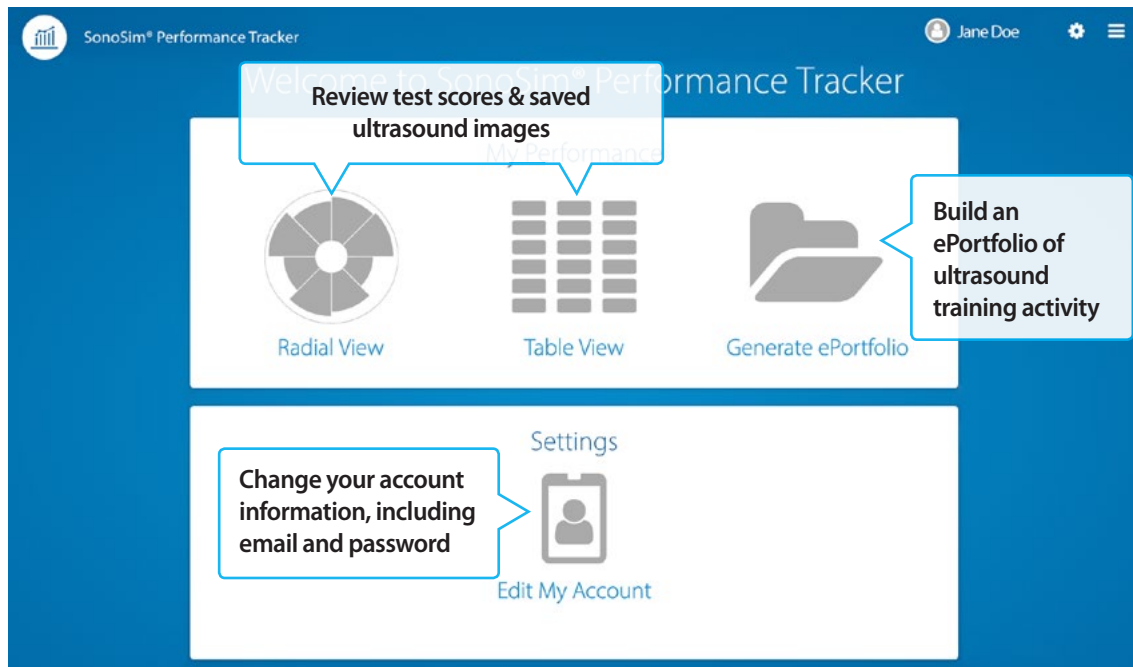

*Not sure where to start? Explore "Generate ePortfolio" to build a PDF of your ultrasound images and Module progress.*

▶ For a video tour of product features, please visit [sonosim.com/videos](https://sonosim.com/videos)

### Need a personal demo?

**Our Member Support Team is ready to assist!**

For a personal demo or help using the features, please contact [support@sonosim.com](mailto:support@sonosim.com) with the subject line “**Product Overview**” or call 855.873.7666 (M-F 6am-6pm, Sat 8am-4pm PT)

### The SonoSim App

**Exclusive for Members**



The Easiest Way to Learn Ultrasonography®

